
Assembly instructions (FY965L)

- 1.Hexagon Screw 2 pcs
- 2.Arc-shaped washer 2 pcs
- 3.Five-star tighten nut 2 pcs


Put in the left and right side push handle tubes into the frame, adjust to the desired height and tighten the nut.
(refer to No. 2 photo)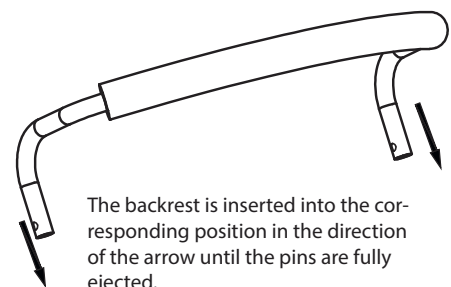

The backrest is inserted into the corresponding position in the direction of the arrow until the pins are fully ejected.
(refer to No. 2 photo)

No. 1 photo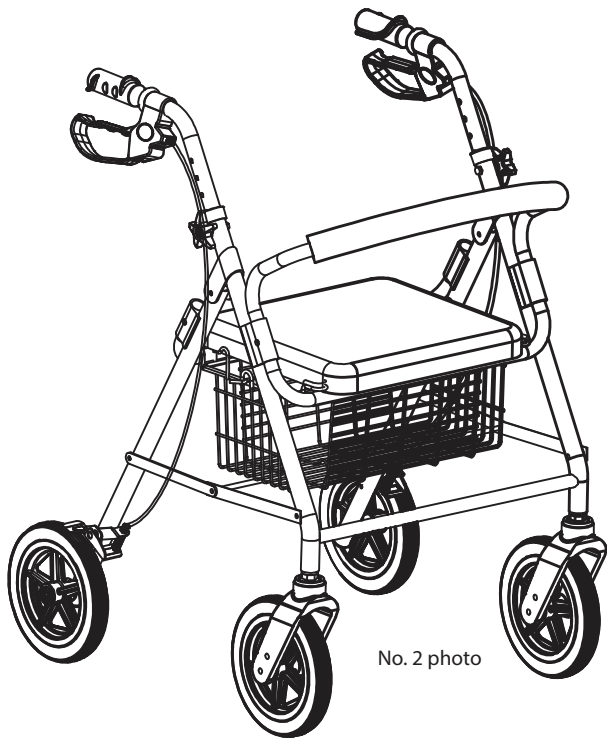

WARNING

- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions. If you are unable to understand the warnings, cautions or instructions, contact a health care professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Each individual should ALWAYS consult with their physician or therapist to determine the proper adjustment and usage.
- DO NOT use the rollator as a wheelchair or transport device. Rollators are not intended to be propelled while a person is seated on it.
- The brakes MUST be in the locked position before using the seat.
- A physical/occupational therapist should assist in the height adjustment of the rollator for maximum support and correct brake activation.
- Care should be taken to ensure that all hand and height adjustments are secure, and that casters and moving objects are in good working order before using this or any mobility aid.
- All wheels MUST be in contact with the floor at all times during use. This will ensure the rollator is properly balanced.
- When using the rollator in a stationary position, the hand brakes MUST be locked.
- The rollator basket has a weight limitation of 11 lbs (5 kg). The pouch has a weight limitation of 10 lbs (5 kg). Items placed in either the basket or the pouch should NOT protrude from the basket or pouch.
- If push handles are exposed to extreme temperatures (above 38°C or below 0°C), high humidity and/or becomes wet, prior to use, ensure the hand grips DO NOT twist on the rollator handle - otherwise damage or injury may occur.
- After installation and before use, ensure that all the attaching hardware is tightened securely.
- DO NOT attempt to reach objects if you have to move forward in the seat. Reaching for these objects will cause a change in the weight distribution of the rollator and may cause the rollator to tip, resulting in injury or damage.
- DO NOT hang anything from the frame of the rollator. Items should be placed in the basket or the pouch.
- The backrest should ALWAYS be in place and is NOT designed to support the total body weight of the user.
- Before attempting to reach objects or pick them up from the floor by reaching down between your knees, place your feet securely on the floor. Use EXTREME caution when reaching for any object.

Maximum load: 150 kg

ACCESSORIES WARNING

If any accessories are missing, broken or damaged, stop using the chair until repaired.

Opening the Rollator

1. Lift the seat
 2. Push down the crossbar for opening the rollator
- NOTE: Ensure the locking mechanism is fully engaged.
3. Install the accessories: B (basket and backrest).

Folding the Rollator

1. If necessary, remove the accessories and lift up the seat.
 2. Lift the locking mechanism on both sides and fold the rollator up.
- NOTE: Pouch under the seat

Pouch

Attach the pouch (not shown) under the seat.

Basket

Install the basket onto the front seat support tube so the hooks rest on the front tube or underneath the seat.

Care and Maintenance

1. Verify the proper operation of the brakes and have them adjusted, if necessary.
2. If the hand grip is loose, DO NOT use the rollator. Tighten the hand grips.
3. Inspect the wheels periodically for wear or damage.
4. Replace any broken, damaged or worn items immediately.
5. Periodically inspect the casters and caster stems for assembly tightness and verify that the wheels are free of hair, lint and other debris that could interfere with free wheel operation.

Assembly, Installation and Operating Instructions


Using the Hand Brake

1. Pull the top portion of the brake handles up towards the push handles.
2. Perform one of the following:
 - To Remain Stationary - Hold the brake handle up.
 - To Continue Mobility - Release the brake.
3. To lock, push down on the bottom portion of the brake handle.
4. To unlock, pull up on the top portion of the brake handle.

Adjusting the Height of the Rollator

NOTE: After adjusting the height, the rollator's brakes must be checked and, if necessary, adjusted.

1. Either loosen the tube tightening levers or remove the adjustment knob by turning it counterclockwise.
2. Position the push handle so that when the user's arm is down to their side the hand grip is at wrist height.
3. Either tighten the tube tightening lever or install the adjustment knob into one of the six adjustment holes by turning clockwise.
4. Repeat STEPS 1-3 for the other side.

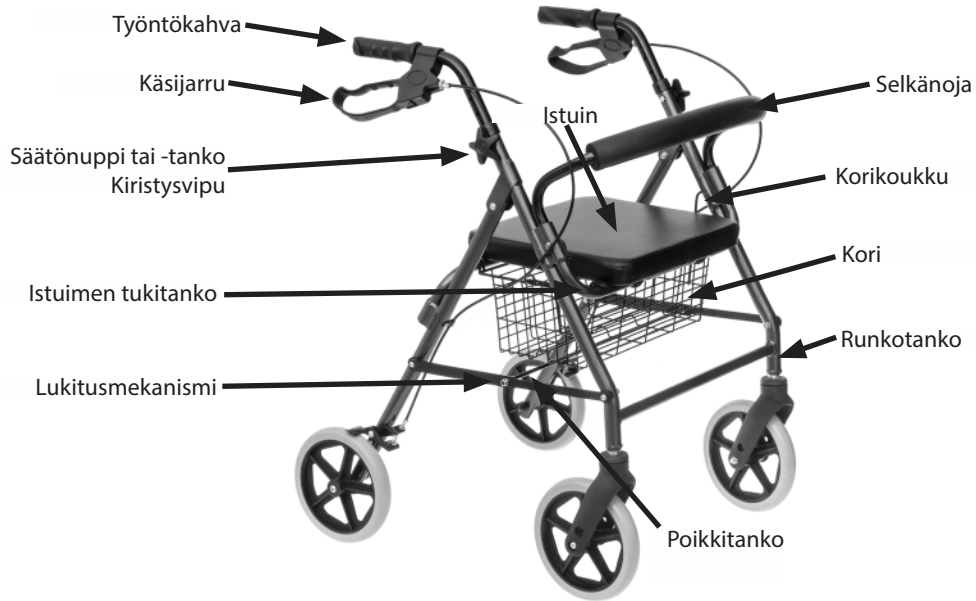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